

gonasan SPORT OA Knee Brace

Knee Brace for relief using the three-point principle, adjustable

Indications

- Early to moderate varus gonarthrosis
- Genu varum
- Early to moderate valgus gonarthrosis
- Genu valgum
- Mild knee instability

Features

- Physiological knee position thanks to 3-point principle
- Elastic knitted mesh provides stabilisation and cushioning
- Easy positioning thanks to patella cut-out and non-elastic Velcro fasteners
- Adjustable extension and flexion stops on the ROM hinge
- Low weight and slim design due to restriction to functional components

Mode of operation

The gonasan SPORT OA Knee Brace is a soft orthosis that uses the 3-point principle to change the flow of force in the knee joint. The redistribution of load prevents further reduction of the knee joint space and relieves pressure on the painful area. Light compression of the tissue surrounding the knee joint subconsciously reminds the user to put the correct amount of weight on their knee joint (proprioception). At the same time, the knee is stabilised, guided and limited if necessary.




Size chart and article numbers

Size	One size	
Circumference at thigh [cm] (see diagram)	40 - 50	
Circumference at calf [cm] (see diagram)	31 - 41	
Art.-No. left*	varus on the left leg (lateral relief)	400-L
	valgus on the right leg (medial relief)	
Art.-No. right**	valgus on the left leg (medial relief)	400-R
	varus on the right leg (lateral relief)	

* A left orthosis is suitable for lateral gonarthrosis on the left leg and medial gonarthrosis on the right leg

** A right orthosis is suitable for lateral gonarthrosis on the right leg and medial gonarthrosis on the left leg

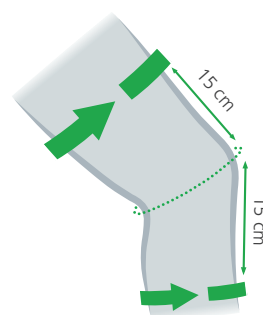

Dimension sheet lower extremity

For orders please copy and send filled in form to:
E-Mail: info@albrechtgmbh.com
Fax: 08051-961 29 30

Order Quotation

Date: _____

Medical supply store: _____

Customer ID: _____

Orthopaedic technician: _____

Patient ID: _____

Side

Left Right

Product name

- Hypex® Lite Knee Brace
- Jack PCL Brace
- CDS® Knee Brace
- CDS® Lock Knee Brace
- CDS® Knee Brace Amputation
- CDS® Ankle Brace
- CDS® Ankle Brace Neuro
- CDS® Combination Brace Knee-Ankle
- CDS® lock Combination Brace Knee-Ankle
- other: _____

Direction of spring force (for CDS®)


Knee Extension Flexion

Ankle Dorsal extension Plantar flexion

For children

Colour

- Green 
- Blue 
- Orange 
- Pink 

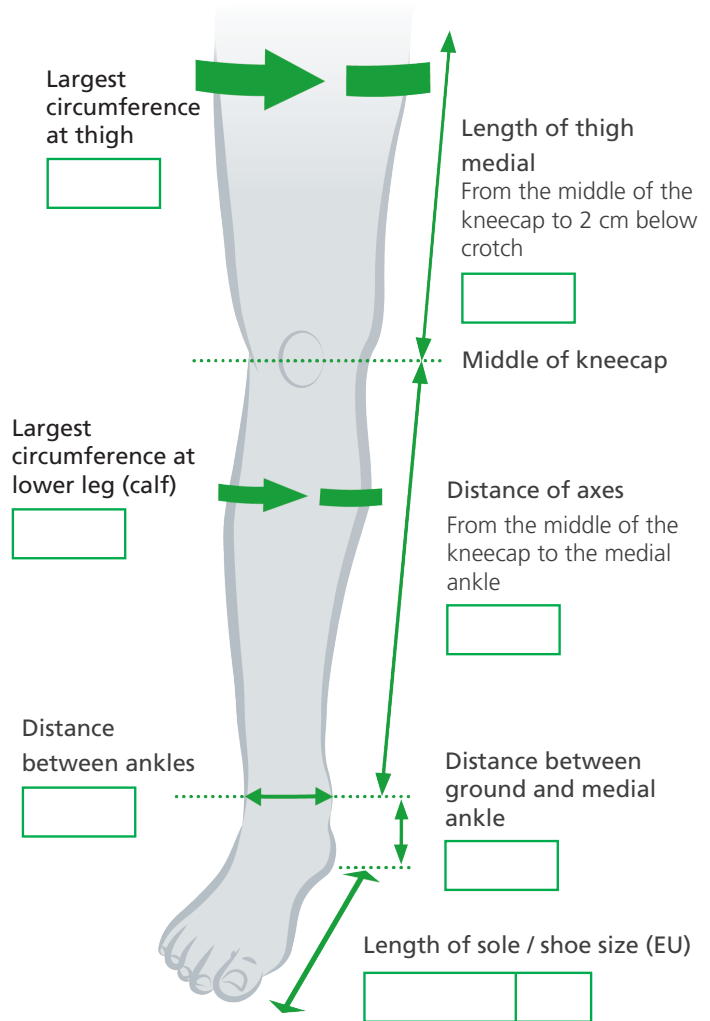

	Distance of axes
CDS® Combinationbrace Knee-Ankle JUNIOR (Dynamic orthoses)	12 - 38 cm
CDS® lock Combinationbrace Knee-Ankle JUNIOR	12 - 31 cm
CDS® Combinationbrace Knee-Ankle (Dynamic orthoses)	29 - 51 cm
CDS® lock Combinationbrace Knee-Ankle	29 - 51 cm

Technical implementation (to be filled in by albrecht)

- Size modification
- Modular adaptation

Art.-No.: _____

Approval: _____