CDS® SPRUNGGELENKSCHIENE

CDS® SPRUNGGELENKSCHIENE JUNIOR

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User Instructions

Contents

1. Introduction	4
1.1. Foreword	4
1.2. Customer information	4
1.3. Mode of operation	4
1.4. Application	4
1.5. Scope of delivery	5
1.6. Declaration of conformity	5
1.7. Features	5
1.8. Indications	6
1.9. Contraindications	6
1.10. Safety instructions	6
2. Adjustment by the orthopaedic technician	8
2.1. Adjustment to the patient	8
2.1.2. Setting of shell elements for sizes L/L, L/M, L/S, M/S	10
2.1.3. Adjusting the textile shells for sizes S/S, XS/XS, XXS	11
2.1.4. Adjusting the distance of the joint supports	12
2.1.5. Adjust the foot rotation	14
2.1.6. Adjusting the ankle axis	15
2.2. Adjusting the hinge	16
2.2.1. Adjusting the position of the redression and limitation range for size L/L, L/M, L/S, M/S	16
2.2.2. Adjusting the position of the redression area for sizes S/S, XS/XS, XXS	18
2.2.3. Setting the limitation for size L/L, L/M, L/S, M/S	20
2.3. Fitting the brace by the orthopaedic technician	21
2.3.1. Deactivation of spring tension	21

EN

User Instructions

2.3.2. Attach the brace to the leg	22
2.3.4. Final adjustment.....	25
2.3.5. Optional setting of the spacer for size L/L, L/M, L/S, M/S	25
2.3.6. Activate the spring tension	26
2.3.8. Changing the spring tension	29
3. Handling by the patient	30
3.1. Removing the brace	30
3.2. Putting on the brace.....	32
4. Cleaning, maintenance and disinfecting	34
4.1. Pads and straps.....	34
4.2. Rods (hinges).....	34
5. Technical data / material	35
6. Size chart and article numbers	35
7. Transfer of the brace	37
8. Disposal.....	37
Duty to report.....	37

CDS® Ankle Brace | Junior

User Instructions

EN

1. Introduction

1.1. Foreword

Dynamic spring-loaded orthoses can be used to treat joint contractures caused by both neurological and orthopaedic conditions. The shortening of the tissue surrounding the joint decreases the range of motion affecting the patient's everyday life. Therefore, our CDS® concept aims to increase the range of motion without pain by applying a constant, appropriate low load prolonged stretch (LLPS).

1.2. Customer information

For your own safety please read through these User Instructions carefully and accurately before using the brace. The instructions, notes and procedures must be read and understood thoroughly in order to benefit from the correct operation and use of the device. If anything in the User Instructions is not clear, or any instructions, operating procedures or safety information is not fully understandable, please contact the appropriate specialist retailer or albrecht GmbH directly, before you use the brace. This particularly applies to the safety instructions.

1.3. Mode of operation

The CDS® Ankle Brace | CDS® Ankle Brace Junior is based on the CDS®-principle and has been designed to treat an extension deficit of the ankle. The brace applies a dynamic low load prolonged stretch to stimulate growth in the contracted tissue. **The adjustable redression range protects the tissue from painful overstretching.**

1.4. Application

The brace is only used for the orthotic treatment of the ankle joint and is only intended for contact with intact skin.

Only for short distances in the house and for therapeutic purposes. A decision about this lies with the physician treating the patient.

Orthoses are not suitable for strengthening or replacing damaged muscles!

Size:	Body weight
S/S, XS/XS, XXS:	up to 50 kg
M/S	up to 75 kg
L/L, L/M, L/S	up to 100 kg

User Instructions

1.5. Scope of delivery

The CDS[®] Ankle Brace | CDS[®] Ankle Brace Junior is delivered to you ready to use in the size requested, with User Instructions and labeling on the product.

Please check the completeness of the brace at delivery.

- Brace with padding and straps
- albrecht GmbH hexagon key
- User Instructions

1.6. Declaration of conformity

The albrecht GmbH company, as the manufacturer solely responsible, declares that the CDS[®] Ankle Brace, CDS[®] Ankle Brace Junior conforms to the Regulation (EU) 2017/745 concerning medical devices.

1.7. Features

- Therapy in dorsal extension
- Spring force individually adjustable
- Activation and deactivation of the redression force without the use of tools and without changing the set spring force
- Optimum length adjustment by means of individually movable shells on the lower leg
- Comfortable to wear due to exact circumferential adjustment by means of 2-way adjustable Velcro fasteners and light, air-permeable aluminium shells
- Adjustment of the redression range in dorsal extension in 15° steps (size S/S, XS/XS, XXS)
- Height adjustment in the ankle joint with individual adjustment of the ankle joint axis
- Adjustable foot rotation: 5° external rotation, 5° internal rotation
- **CDS[®] Ankle Brace L/L, L/M, L/S:**
 - Comfortable to wear due to exact circumferential adjustment by means of 2-way adjustable Velcro fasteners and light, air-permeable aluminium shells
 - Sole with roll-off aid
- **CDS[®] Ankle Brace S/S, XS/XS, XXS:**
 - Adjustment of the flexion range in 15° steps
 - High wearing comfort due to air-permeable, light, textile shell elements

CDS® Ankle Brace | Junior

User Instructions

1.8. Indications

The physician will prescribe the type of treatment to apply based on his or her diagnostic findings.

Generally, the use is indicated in:

- Joint contraction, among other things, as a result of:
 - vegetative state
 - cerebral palsy
 - genetic syndromes that are clinically associated with tonus elevation in the lower limb
 - epilepsy
 - spina bifida
 - near-drowning
 - surgical procedure
 - burns
 - stroke
 - spinal cord Injury
 - paralysis
 - cranial brain trauma
 - osteoarthritis and chronic polyarthritis
 - joint replacement
- For preventing new contractures after arthrolysis
- After conservative therapy of capsular ligament injuries

For all other indications, the doctor must be consulted.

1.9. Contraindications

- Bony obstruction, osteoporosis
thrombophlebitis

The brace is intended exclusively for contact with intact skin.

1.10. Safety instructions

The optimal effect of the brace is only achieved when used correctly.

- The brace must only be used in the intact, complete and mechanically undamaged condition and with complete and intact cushioning and walers. This must be verified by the user before each usage.
- Opening or removing one or more belts, as well as excessive loosening of the waler when using the brace leads to a reduction of the therapeutic effect of the brace and may lead to injury.
- The brace must not be worn over open wounds.
- The skin should be free of oils, grease, gels or other debris, to prevent reactions with the skin or the structure of the material.
- The orthosis should fit firmly but not too tight, so as not to restrict the blood circulation and adversely affect nerve and lymph vessels. Excessive compression is therefore to be avoided.
- Combination with other products is currently not provided for or is to be agreed with the manufacturer in writing.
- The brace is not intended for single use, but is intended for multiple use by a single person.

User Instructions

- The product as delivered is not sterile.
- Contact your physician immediately in the event of an allergic reaction.
- Please note that cushioned sections can heat up under direct sunlight. Protect the orthosis from direct sunlight if necessary.
- Currently there is no test for flammability. Exercise caution when using the orthosis in the direct vicinity of open flames such as lighters and cigarettes.
- The mechanical functions must only be adjusted using the supplied tools in order to avoid injuries and damage of the hinge.
- When adjusting the hinge rods to the shape of the extremity by using an orthopaedic bending iron, you must not bend the rods in the area of the hinge housing or the hinge cover as this could lead to damage or break of the hinge.

1.11. Warranty

In addition to the legal warranty, we provide a 6-month durability guarantee for the orthosis. If properly used, this guarantees that the orthosis will function without fault. This excludes the padding and straps, which are usually liable to a certain amount of wear and tear. This kind of wear and tear does not represent a product defect. This manufacturer's warranty is subject to the condition that the orthosis is used as a medical rehabilitation device and for no other purpose than that described in the instructions for use. Changes to the orthosis or the removal / damage to the quality management seal will invalidate the warranty.

CDS® Ankle Brace | Junior

User Instructions

2. Adjustment by the orthopaedic technician

2.1. Adjustment to the patient


- Our CDS® Braces are constructed to be adjustable.
- The position of the shell components can be changed and they can be shaped.
- The hinge rods can be adapted to the shape of the extremity by using an orthopaedic bending iron.
- The strap lengths can be adjusted to different girths and shortened if necessary.

EN

User Instructions

2.1.1. Adjusting the brace by using an orthopaedic “bending iron”

The brace is anatomically contoured. However, if a different shape is required, the hinge rods can be adjusted to the shape of the patient’s leg with the aid of an orthopaedic bending iron. Loosen the screws on the shell components and either move or remove them. Then adjust the hinge rods to the shape of the extremity.

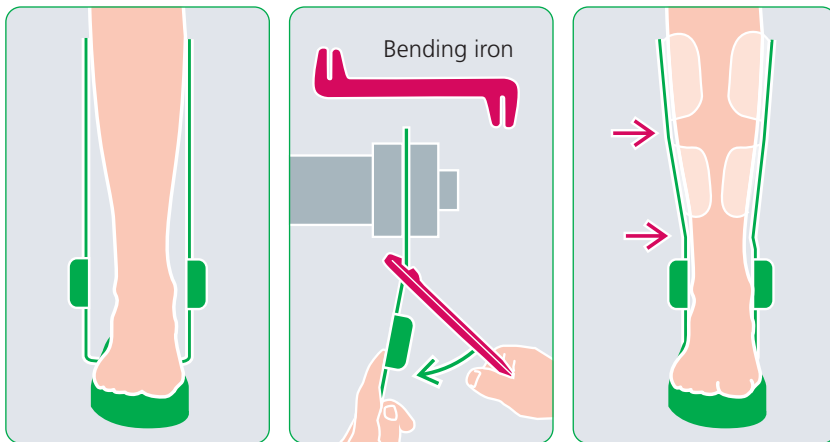When adjusting the hinge rods to the shape of the extremity by using an orthopaedic bending iron, you must not bend the rods in the area of the hinge housing or the hinge cover as this could lead to damage or break of the hinge.

CDS® Ankle Brace | Junior

User Instructions

2.1.2. Setting of shell elements for sizes L/L, L/M, L/S, M/S

The shell elements can be moved.

- 1 Loosen the screws on the shell elements with the tool supplied without unscrewing them completely.
- 2 Move the shell elements to the desired position.
- 3 Tighten the clamping screws again.

EN

The shell elements can be adapted to the contour of the limb by bending them.

User Instructions

2.1.3. Adjusting the textile shells for sizes S/S, XS/XS, XXS

The textile shells can be adjusted up to 2.5 cm in height.

- 1 Loosen the screws on the shell elements with the tool supplied.
- 2 Remove the textile shell.
- 3 Using a punch tool, punch another hole vertically below the existing bolt hole (max. 2.5 cm).
- 4 Slide the shell over the support and secure it in the new screw hole.

Please note that for the sizes S/S, XS/XS, XXS the design of the lower leg shell consists of a textile shell.

CDS® Ankle Brace | Junior

User Instructions



2.1.4. Adjusting the distance of the joint supports

You can change the distance of the hinges from the edge of the sole.

- 1 Remove the sole.
- 2 Loosen the screws in the sole plate.
- 3 Move the rods.
- 4 When the optimal position for both hinges is found, tighten the screws again.

EN

Size L/L, L/M, L/S

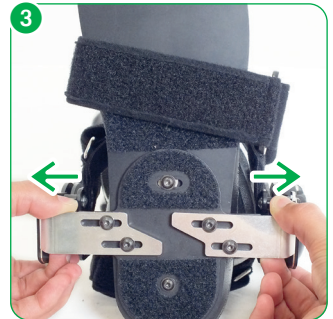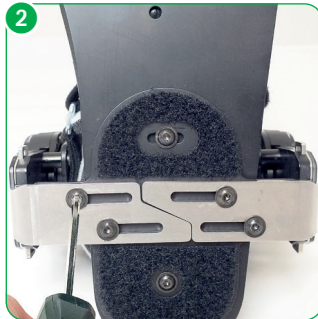User Instructions

Size M/S, S/S, XS/XS, XXS:

The sole part of the M/S, S/S, XS/XS, XXS brace is designed so that it can be suitably adjusted by the orthopaedic technician. The footbed maybe made individually by the specialist trade to ensure an optimal bedding of the foot.


CDS® Ankle Brace | Junior

User Instructions


2.1.5. Adjust the foot rotation

- 1 Remove the sole.
- 2 Loosen the screws in the foot plate and adjust the foot plate to the desired position.
- 3 Tighten the two clamping screws again.

Size L/L, L/M, L/S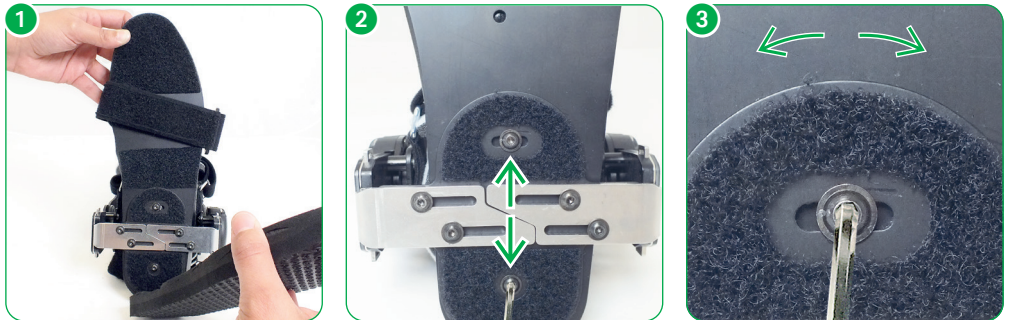

Size M/S, S/S, XS/XS, XXS:

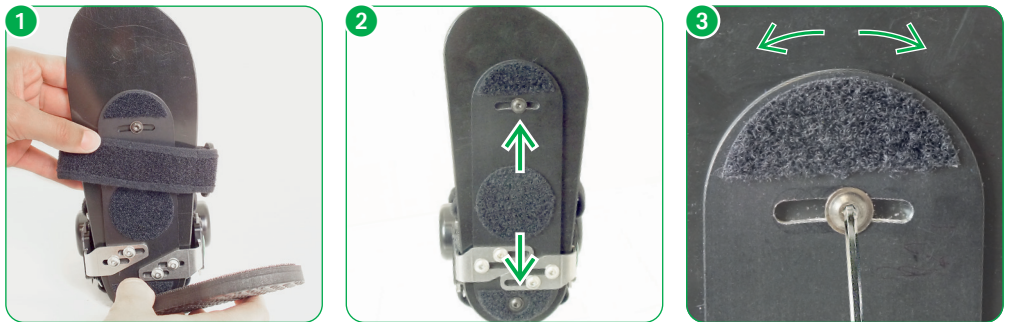User Instructions

2.1.6. Adjusting the ankle axis

- 1 Pull the latch up to the stop.
- 2 Slide the joint to the desired height (see scale).
- 3 Snap the latch into place.

Size L/L, L/M, L/S, M/S

EN

Size S/S, XS/XS, XXS:

The lateral and medial sides can be adjusted separately, so that the hinges can be individually adjusted to the ankle axis.

CDS® Ankle Brace | Junior

User Instructions

2.2. Adjusting the hinge

2.2.1. Adjusting the position of the redression and limitation range for size L/L, L/M, L/S, M/S.

The CDS® Ankle Brace is constructed on the basis of a 360° hinge. In a 360° hinge, the spring housing and the lower leg rod can be pivoted against each other. The position of the redression and limitation range can be thereby altered and adjusted to the patient.


EN

In neurological patients it may be necessary to bring the redression range of the CDS® Ankle Brace further into plantar flexion than the factory setting. The position of the stops depends on the position of the redression and limitation range.

User Instructions

- 1 Remove the pad.
- 2 Remove the two screws.
- 3 You can adjust the position of the redression range in 15° steps. Use a goniometer to determine the desired position and adjust the hinge accordingly.
- 4 Insert the screws again and tighten them.
- 5 Insert the pad again.

The setting must be the same for both hinges.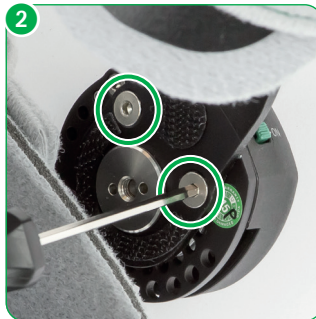

CDS® Ankle Brace | Junior

User Instructions




2.2.2. Adjusting the position of the redression area for sizes S/S, XS/XS, XXS

The CDS® Ankle Brace Junior in sizes S/S, XS/XS and XXS is built on the basis of the 360° Nano Hinge. In the 360° Nano Hinge, the position of the spring housing to the lower leg rod can be adjusted. This allows the location of the redression area to be changed and adapted to the patient.

- 1 Remove the pad on the back of the hinge.
- 2 Loosen the central screw with the tool supplied until the hinge rod can be lifted completely from the tothing.
- 3 You can adjust the position of the redression range in 10° steps. Use a goniometer to determine the desired position and adjust the joint accordingly. By changing the redression range, the extension or flexion stop can be adjusted in 10° increments. The joint settings must be made identically on both sides.
- 4 Place the joint rod back onto the gear teeth and tighten the central screw again.
- 5 Put the pad back on again

EN

User Instructions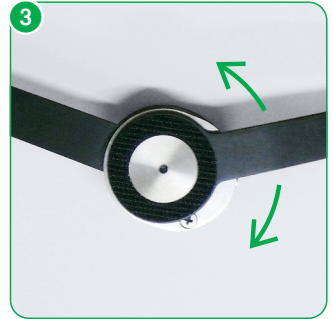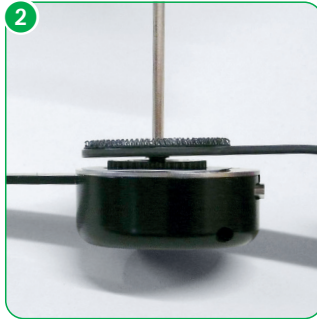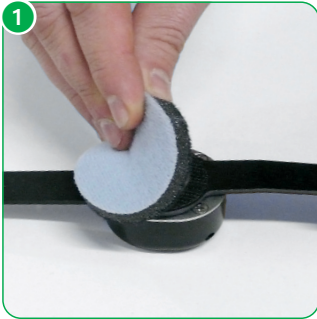

EN

CDS® Ankle Brace | Junior

User Instructions

2.2.3. Setting the limitation for size L/L, L/M, L/S, M/S


The limitation restricts the range of motion. All steps must be executed identically with both hinges.

Dorsal extension and the redression range are limited by the stop screw. In a second step, the set value can be reduced up to a maximum of 15°, enabling infinitely variable adjustment of the redression range.

Before setting the dorsal extensions limitation, you must deactivate the spring tension.

EN

- 1 Turn the green switches on both hinges to „off“.
- 2 Bring the brace into plantar flexion.
- 3 There are three holes on the side of the CDS® housing. The number of degrees depends on the position of the redression and limitation range and can be determined with a goniometer. The stop screw is located in one of the holes. Then turn the stop screw with the supplied tool from the CDS® housing.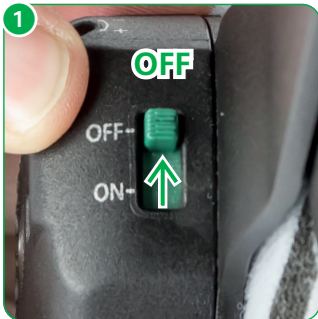

Before screwing it into one of the three holes, bring the brace into plantar flexion, so that the holes are freely accessible.

Position the stop screw in the desired position and tighten the screw. The limitation can be adjusted to the treatment progress with increasing mobility of the patient.

Please be aware that the brace must only be used with the stop screw positioned and tightened in one of the holes, otherwise the hinge will be damaged.

User Instructions

2.3. Fitting the brace by the orthopaedic technician

2.3.1. Deactivation of spring tension


Before fitting the brace, you must deactivate the spring tension.

- 1 Set the switches on both hinges to "off".
- 2 Place the brace into plantar flexion as far as the patient is allowed to move. The activation mechanism of the hinge is then set to this position and the patient can reactivate the spring force in this position.

Size L/L, L/M, L/S, M/S

Size S/S, XS/XS, XXS: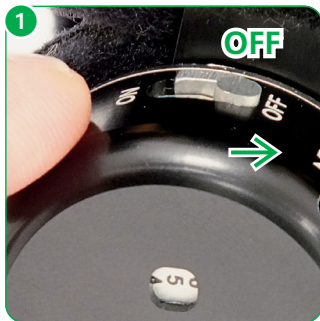

CDS® Ankle Brace | Junior



User Instructions

2.3.2. Attach the brace to the leg

To make it easier to attach the brace to the patient, adjust the length of the lower leg straps to the maximum length without unthreading them.


- 1 Open the instep strap and the toe strap.
- 2 Loosen the upper and lower tibia straps.
- 3 The easiest way to put on and adjust the straps is to sit down. The patient steps into the brace with his foot from the front. You have the option of adjusting the lower leg rods to the patient's leg contour by bending them.

Size L/L, L/M, L/S, M/S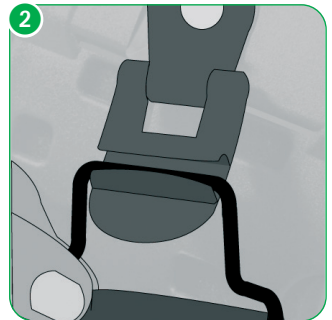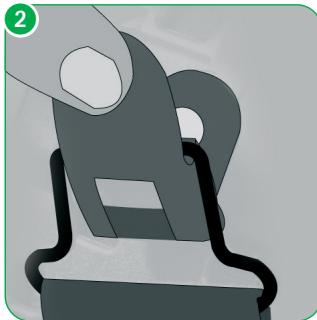

User Instructions

Size S/S, XS/XS, XXS:

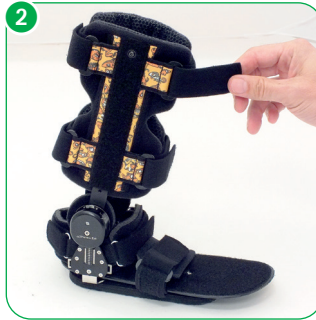Make sure that the joints are as parallel as possible to each other to ensure wear-free operation of the brace.

EN

CDS® Ankle Brace | Junior

User Instructions

2.3.3. Adjust strap lengths as necessary

Adjust the strap to the desired length and shorten it if necessary containing the doubled sided hook end.

- 1 First fasten the heel strap. This prevents the foot from slipping dorsally. Adjust the length of the strap so that it sits snugly on the heel.
- 2 Fasten the instep strap.
- 3 Fasten the toe strap.
- 4 Close the leg straps in the following order:
 - Lower calf strap
 - Lower shin strap
 - Upper calf belt
 - Upper shin belt

If necessary, the additional strap pads provided can be installed under the straps.

Size L/L, L/M, L/S, M/S

Size S/S, XS/XS, XXS:

User Instructions

2.3.4. Final adjustment

After fastening the individual straps, check that the straps are the correct length and that the brace is in the correct position, and correct if necessary. Ensure that the straps are not too tight so as not to interfere with the circulation.

The shell elements are anatomically contoured. You can also shape the shell elements with the hand to the leg contour directly on the patient.

2.3.5. Optional setting of the spacer for size L/L, L/M, L/S, M/S

If necessary, the suture or scar spacer can be attached to the instep strap as well as the heel strap.

CDS® Ankle Brace | Junior

User Instructions

2.3.6. Activate the spring tension

- 1 To activate the spring tension, turn the green switches on both hinges to "on".
- 2 Bring the brace into plantar flexion until you feel a slight resistance that you have to overcome in order to activate the spring tension.

Size L/L, L/M, L/S, M/S

EN

Size S/S, XS/XS, XXS:

User Instructions

2.3.7. Setting the spring tension to the intensity needed by the patient


- 1 The spring tension setting is displayed on the CDS® housing by a scale from 0 to 15. The ranges above 15 and below 0 are marked in red. **To prevent damage to the CDS® hinge, the red range in the CDS® hinge window must be avoided.**
- 2 Insert the tool as far as it will go into the side hole on the hinge. By turning clockwise or towards + the spring tension is increased and it is decreased by turning anticlockwise or towards -.

The intensity of the spring tension must be equal in both hinges.

The intensity of the spring tension is not altered by activation or deactivation of the spring tension.

The spring tension may be adjusted only in consultation with the treating physician.


Size L/L, L/M, L/S, M/S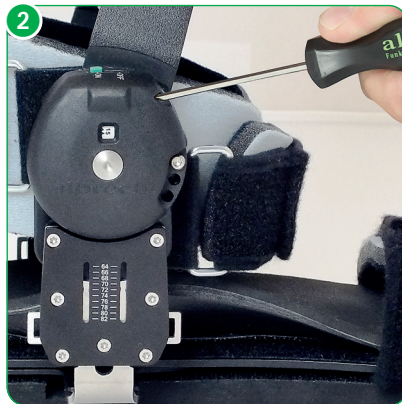

CDS® Ankle Brace | Junior


User Instructions

Size S/S, XS/XS, XXS: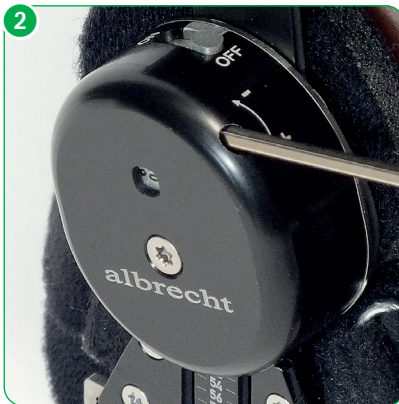

EN

Torque in Nm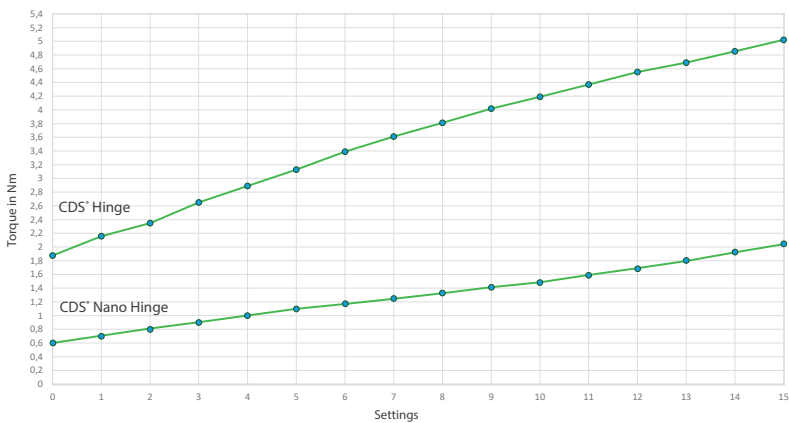

CDS® Hinge (size L/L, L/M, L/S)

CDS® Nano Hinge (size S/S, XS/XS, XXS)

User Instructions


2.3.8. Changing the spring tension

The spring tension can be adjusted according to the to treatment progress.

Insert the tool as far as it will go into the side hole on the hinge. By turning clockwise or towards + the spring tension is increased and it is decreased by turning anticlockwise or towards -.

The strength of the spring tension must be the same for both hinges.

Size L/L, L/M, L/S, M/S

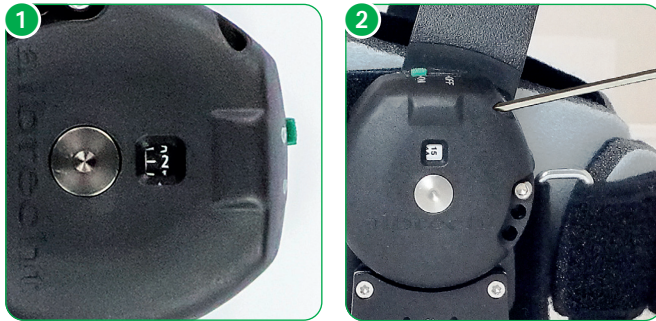Size S/S, XS/XS, XXS:

The spring tension may be adjusted only in consultation with the treating physician.

CDS® Ankle Brace | Junior

User Instructions

3. Handling by the patient


3.1. Removing the brace

Before removing the brace you must deactivate the spring tension.


- 1 To do so, turn the green switches on both hinges to "off".
- 2 Bring the brace into plantar flexion.

EN

Size L/L, L/M, L/S, M/S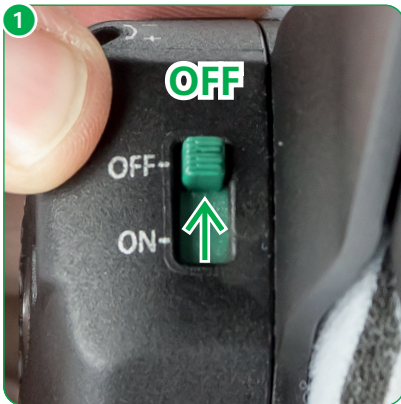


Size S/S, XS/XS, XXS

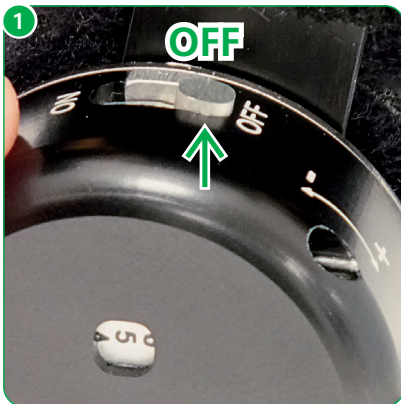User Instructions

- 1 First, loosen the toe strap.
- 2 Then loosen the instep strap.
- 3 Open the clips off the upper and lower shinstraps and do not unthread the straps. Step out of the brace in front with the foot.

Size L/L, L/M, L/S, M/S

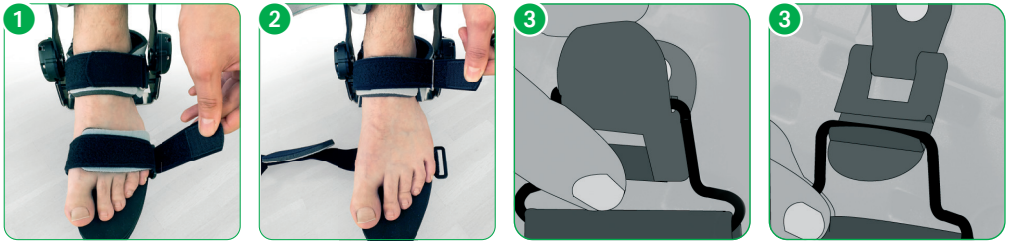Size S/S, XS/XS, XXS

CDS® Ankle Brace | Junior


User Instructions

3.2. Putting on the brace

Attaching and adjusting the straps is easiest when sitting.

- 1 Step into the brace from in front with the foot.
- 2 Fasten the instep strap.
- 3 Fasten the toe strap.
- 4 Then fasten the lower calf strap.
- 5 Fasten the upper calf strap.
- 6 To activate the spring tension, turn the green switches on both hinges to "on".
- 7 Bring the brace into plantar flexion until you feel a slight resistance that you have to overcome in order to activate the spring tension.

Size L/L, L/M, L/S, M/S

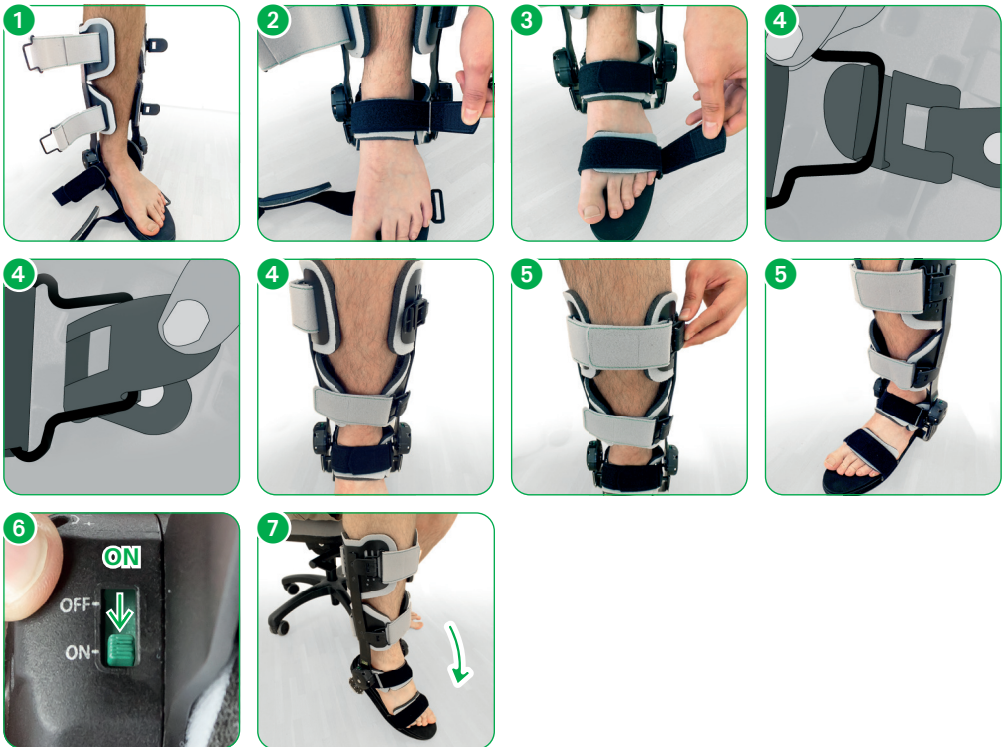User Instructions

Size S/S, XS/XS, XXS

EN

We wish you a successful treatment and are always available to answer any questions.

CDS® Ankle Brace | Junior

User Instructions

30° Hand Wash

Do Not Bleach

Do Not Iron

No Dry Clean

Do Not Tumble Dry

4. Cleaning, maintenance and disinfecting

The orthosis is designed to be maintenance-free. To ensure proper operation over the period of treatment the orthosis should be cleaned regularly (at least every 3 months) or as required, according to the following instructions.

4.1. Pads and straps

- All fabrics can be washed by hand at 30 °C using water and a mild detergent and/or disinfectant.
- Not machine washable.
- In the case of more severe soiling, a replacement set of textile parts is available.

4.2. Rods (hinges)

- Clean all parts of the brace with a wet cloth soaked with water and a mild detergent and/or disinfectant.
- Wipe down surfaces with a cloth soaked with disinfectant.
- Wet completely, and do not wipe off.
- Spray inaccessible surfaces.
- When spraying ensure complete wetting.
- A mild alcohol-based disinfectant is recommended.

Ask your physician or pharmacist when selecting a disinfectant, and follow the instructions given by the disinfectant manufacturer. The Robert Koch list of approved disinfectants can be found at www.rki.de.

User Instructions

5. Technical data / material

Name	Material
Weight	730 g - 1,264 g
Padding material	PU foam with PA hook and loop velour (size L/L, L/M, L/S, M/S) Siliconised fastener (size S/S, XS/XS, XXS)
Strap material	PA strap with PA hook and loop velour
Brace material	aluminium
Sole	EVA foam

6. Size chart and article numbers

Name	Length of thigh shell medial ¹	Circumference of calf	EU-shoe size	Length of sole ²	Art. No. left	Art. No. right
Ankle L/L	29.5 - 33 cm	32 - 47 cm	44 - 47	30 cm	920LL-L	920LL-R
Ankle L/M	29.5 - 33 cm	32 - 47 cm	40 - 43	27.5 cm	920LM-L	920LM-R
Ankle L/S	29.5 - 33 cm	32 - 47 cm	37 - 39	26 cm	920LS-L	920LS-R
Ankle M/S	20.5 - 24.5 cm	22 - 32 cm	35 - 36	24.5 cm	920MS-L	920MS-R
360° Nano Hinge: Name	Length of thigh shell medial ¹	Circumference of calf	EU-shoe size	Length of sole ²	Art. No. left	Art. No. right
Ankle S/S	20 cm	22 - 32 cm	30 - 34	23 cm	920SS-L	920SS-R
Ankle XS/XS	18 cm	20 - 30 cm	26 - 29	20 cm	920XSXS-L	920XSXS-R
Ankle XXS	15 cm	17 - 22 cm	20 - 25	17.5 cm	920XXS-L	920XXS-R

¹ Measured from the ankle axis

² The foot bedding (insole) has to be made individually by the specialised trade in order to guarantee an optimal bedding of the child's foot. The special construction of the sole part of the paediatric CDS® Ankle Brace facilitates an individual adjustment through the orthopaedic technician.

CDS® Ankle Brace | Junior

User Instructions

Name	Length of thigh shell medial ¹	Circumference of calf	EU-shoe size	Length of sole ²	Art. No. left	Art. No. right
Ankle Junior Green	20 cm	22 - 32 cm	30 - 34	23 cm	920SS-GL	920SS-GR
Ankle unior Blue	20 cm	22 - 32 cm	30 - 34	23 cm	920SS-BL	920SS-BR
Ankle Junior Orange	20 cm	22 - 32 cm	30 - 34	23 cm	920SS-OL	920SS-OR
Ankle Junior Pink	20 cm	22 - 32 cm	30 - 34	23 cm	920SS-PL	920SS-PR
Ankle Bambini Green	18 cm	20 - 30 cm	26 - 29	20 cm	920XSXS-GL	920XSXS-GR
Ankle Bambini Blue	18 cm	20 - 30 cm	26 - 29	20 cm	920XSXS-BL	920XSXS-BR
Ankle Bambini Orange	18 cm	20 - 30 cm	26 - 29	20 cm	920XSXS-OL	920XSXS-OR
Ankle Bambini Pink	18 cm	20 - 30 cm	26 - 29	20 cm	920XSXS-PL	920XSXS-PR
Ankle Malinki Green	15 cm	17 - 22 cm	20 - 25	17,5 cm	920XXS-GL	920XXS-GR
Ankle Malinki Blue	15 cm	17 - 22 cm	20 - 25	17,5 cm	920XXS-BL	920XXS-BR
Ankle Malinki Orange	15 cm	17 - 22 cm	20 - 25	17,5 cm	920XXS-OL	920XXS-OR
Ankle Malinki Pink	15 cm	17 - 22 cm	20 - 25	17,5 cm	920XXS-PL	920XXS-PR

¹ Measured from the ankle axis

¹ The foot bedding (insole) has to be made individually by the specialised trade in order to guarantee an optimal bedding of the child's foot. The special construction of the sole part of the paediatric CDS® Ankle Brace facilitates an individual adjustment through the orthopaedic technician

7. Transfer of the brace

The brace is not intended for single use, but rather is intended for multiple use by a single person. We do not recommend transfer to other users. Should this be desired however, please ensure to pass on the care and cleaning instructions and have the brace checked by an authorized specialist dealer for safe and proper operation.

8. Disposal

The brace contains recyclable materials without toxic or other harmful substances or other environmentally hazardous substances. Provided it is not contaminated with infectious germs, the brace can be deposited in the normal waste disposal. To be sure, consult your specialist orthopaedics dealer.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to the manufacturer and the responsible authorities. Please find our contact details on the back of this brochure.

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